


Linking OCULUS Keratograph[®] to WAVE

Step 1:

Open the OCULUS Keratograph[®] software, select a Patient, and open a previous examination.

Step 2:

Go to **Settings**, then select **Miscellaneous Settings**.

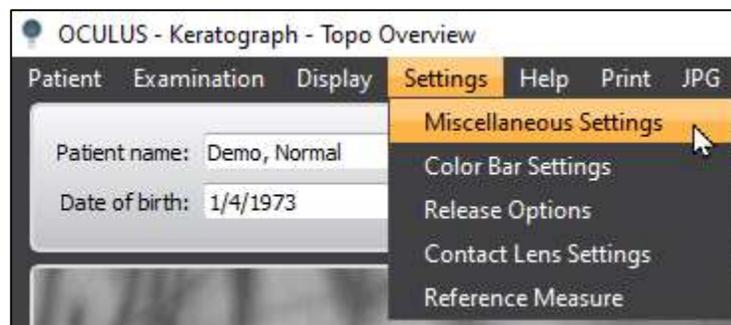


Step 3:

A new window opens. Go to **Export** tab. Under **External Software** select Label: **Wave** from the dropdown menu and check the **Active** box.



If you need help installing the WAVE software or linking it to your corneal topographer, please contact WAVE technical support at 1-855.655.2020.


Step 4: Check the **Start Application** box and navigate to the **WaveChoice.exe** application. Select and then click on **Open** button.

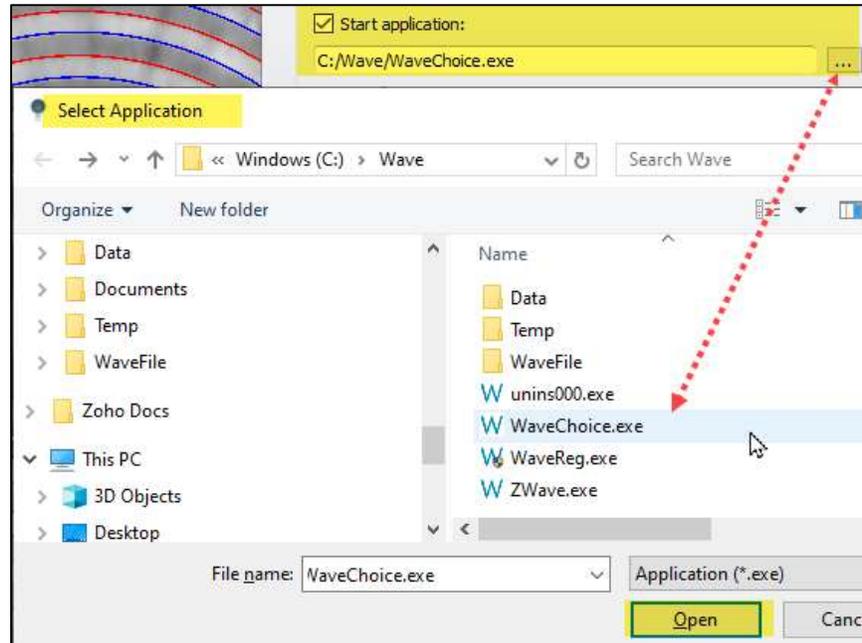

Step 5: In the **Commandline parameter** box type in **Keratograph** (capital letter **K**). Set **Data will be exported to directory** as **C:\Wave\Data**. Make sure **the box is checked** for **Remove old files before export** and that the **Zip-File Format** box is **unchecked**.



Step 6: Click on the **Save** button to complete.

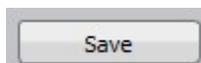

If you need help installing the WAVE software or linking it to your corneal topographer, please contact WAVE technical support at 1-855.655.2020.